LUXLINE 45 / 45R / 45D

TR

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1. Introduction

Thank you for preferring BIÇAKCILAR brand.

Each device that bears BIÇAKCILAR brand reflects a production experience of more than 65 years. The reason behind the leading position of BIÇAKCILAR both in domestic and international markets is the company's uncompromising firm stand concerning the basic principles and consistency of the quality policy. "SAFETY" is the most emphasized principle at BIÇAKCILAR. It is of extreme importance that the contents of this user's manual be strictly followed to ensure the safe operation of the device. Otherwise, hazardous situations effecting the user, patient, or the environment may arise.

Important Warning!

1. Please read this manual carefully before installing, assembling, and using the device.
2. It is the responsibility of the customer to inform the related personnel about the content of this user's manual.
3. Before installation and use, be sure to understand all the details given in this user's manual.
4. It's a device intended for supporting diagnosis that failure of device will not cause any risk for patient and treatment.
5. Device isn't intended for use at surgery rooms.
6. This manual should be kept at an accessible place to the user.
7. The electricity wiring of the location at which the device will be installed should be compatible with IEC 60364-7-710.
8. All the functions of the device should be checked before use.
9. The device lifetime is 10 years.
10. When the lifetime of the device and of its accessories is completed, they must be disposed as contaminated medical waste.

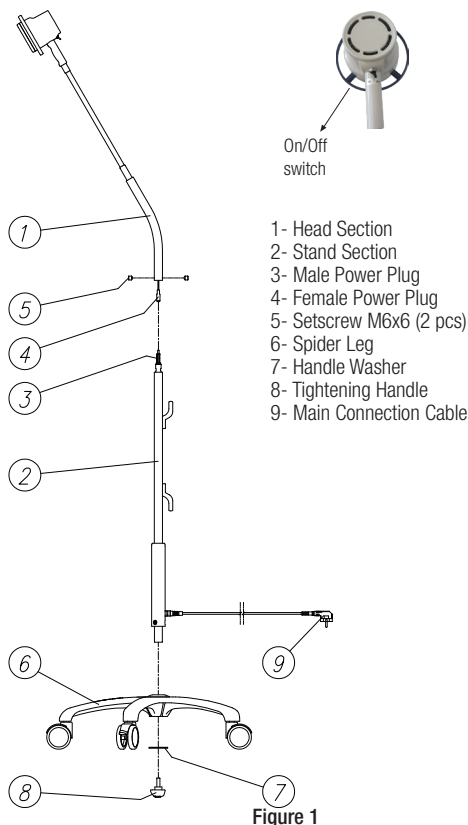
1.1. Purpose of Use

Luxline 45 / 45R / 45D Examination Lamp is used for lightening the body of patient to support diagnosis and treatment.

1.2. User Profiles

It must be used by people employed in their profession and having completed their education in the area of medicine and nursing.

2. Description

3. Packaging, Storage and Transportation

Examination lights are delivered with styrofoam packaging material in carton boxes. Please do not expose the device to any shocks and do not drop it during storage and transportation. The packaging material is recyclable. It should be discarded into the appropriate waste containers.

4. Installation and Placement

4.1. Model Luxline 45;

Take the examination light to the desired place.

- Place the Head Section (1) and the Stand Section (2) on a flat table.
- Connect the Male Power Plug (3) and the Female Power Plug (4).
- Insert the head section (1) and the table section (2) together and secure them with M6x6 setscrew (During this procedure, ensure that the cable winder on the stand line is on the reverse of light movement direction).
- Insert the assembled parts into the housing of the Spider Leg (6) and hand-tighten the clamping handle (8) together with the Handle Washer (7).

Attention!

Do not carry the lamp from its head part. Carry by handling from stand line. Otherwise you have problems with the balance of the device.

4.2. Model Luxline 45R;

Take examination light to where you want to install. Attach rail adapter to the light and secure with tightening handle M6 at the bottom.

4.3. Model Luxline 45D;

Take the examination light to where you want to install. Mark the wall by the aid of the four holes on the wall mounting plate where position you want to install the examination light. Drill a suitable hole for the plastic anchor ($\varnothing 6$) with the drill from the places that you mark.

Do the mounting the examination light with four units cross-recessed pan head tapping screw (5,5x40 mm) and four units plastic anchor ($\varnothing 6$).

5. Use

5.1. Use

ON/OFF switch is provided on the lamp head. Set ON/OFF switch to "ON" position to turn the light on.

- Device must only be used in dry and dust free rooms.
- Do not use damaged luminaries. Damaged cables are also potential risk. Do not place cable to any area close to heat source or onto sharp edges.
- Vent openings (if any) must not be closed during use of luminaries.
- Device must not be used near external heat sources exceeding maximum ambient temperature for device.

Hazard!

Do not attach damaged power cables. If there is any indication for damage on power cable, immediately replace.

Warning!

Supply voltage and frequency must comply with data on model label.

Warning!

Harmful to eyes. Never directly look at light source.

Attention!

Do not use device for long term operations.

5.2. Security Information

1. There shall be at least 50 cm space between the lamp head and the examination point.
2. Leave adequate space in order to ensure that no flammable objects are around.
3. Do not leave the light on without control.
4. Do not crush the power cord and protect it from damage.
5. Do not remove the plug by pulling the cable
6. Since a part of the energy is converted to heat energy, the lamp is also a source of heat.
7. Do not attach damaged power cables. If there is any indication for damage on power cable, immediately replace.
8. Supply voltage and frequency must comply with data on model label.

5.3. Usage Period

The recommended usage period of Bıçakcılar Luxline inspection lamps is 10 years.

5.3. Foreseeable Misuse

BE SURE TO FOLLOW the warnings described in this section as the issues discussed here deal with the wrong uses that may harm the patient, the user, the environment or the instrument itself or clarify the measures the user should take during operation.

- 1. Use the instrument at AC 230V- 50Hz.
(Otherwise the user and the device may be exposed to hazards).
- 2. Do not use your device in oxygen rich and anaesthetically inflammable mixture existing environments.
- 3. Perform the maintenance described in Section 6 on time and in accordance with the instructions.
- 4. Do not use your device other than its intended purpose and under various environmental conditions.
- 5. Harmful to eyes. Never directly look at light source.
- 6. Never place extra weights on top or side of the device.
- 7. Manufacturer cannot be responsible for injuries or damages caused by any use other than intended purpose or failure to comply with safety instructions and warnings.

6. Maintenance

- Power supply cable must be checked for minimum once a year against damage.
- Device doesn't require maintenance. You can procure additional documentation related to this product from manufacturer upon request. These luminaries don't affect other devices. Illumination must be utilized when really needed to save energy.

Warning!

This device, manufactured by Bıçakcılar, is not in the scope of guarantee if not used as indicated in the instructions for use and the maintenance repair or any other intervention is done by personnel other than Bıçakcılar authorized technical service.

6.1. Cleaning

Do not submerge into the water or clean with water because the light cap is not water proof. Do not use abrasive cleaning products. The table surface should be wiped with a wet piece of cloth to remove the disinfectant.

Attention!

Wait for the light to cool before cleaning and unplug the device.

Warning!

Disinfectant sprays may cause corrosion on the mechanical parts or may create explosive mixtures in the components. Therefore, do not use disinfectant sprays.

Caution!

To minimize the risk of infection and comply with these instructions, current occupational health and safety regulations and requirements of national authorities for hygiene and disinfection must be strictly observed.

- Only use disinfectants not to affect functional capacity of the light.
- Solvent, chlorine or abrasive products containing cleaning agents must not be used. These agents might damage plastic parts.
- Cleaning and disinfection agents must be suitable and approved for plastics such as PC, PMMA, PA and ABS.
- Concentrated disinfectants might damage the light.
- Observe specifications on the data sheet of the agent for concentration and effective period.
- Inappropriate clothes might cause scratches.
- Regularly clean the cover to keep it clean.
- Only disinfection by wiping is allowed.
- Clean the PC cover with an appropriate cleaning cloth and relevant cleaning agent.

6.2. Recycling the Device

Do not dispose device with domestic sewage. Dispose the examination light to a waste area or take to a dealer providing relevant disposal service as per local regulations. Cut the cable at the housing.

Above listed products are over 95% recyclable. Examination light is manufactured to comply with recycling, thus majority of the materials used in these products can be recycled or converted into energy at the end of their life cycle. It doesn't include hazardous or to be followed materials.

6.3. Troubleshooting

Device must not be modified or manipulated in any way. Only approved original parts must be used. Any use other than intended use of original parts might cause development of various technical values and life hazard.

Breakdown	Potential cause	Troubleshooting	User profiles
Light doesn't turn on	Contact problem	Turn I/O switch on again	All
Light doesn't turn on	No network voltage	Check network connections and voltage	Electrician
Light doesn't turn on	Defective LED module	Contact with manufacturer's support	Bıçakcılar authorized technical service

6.4. Disposal

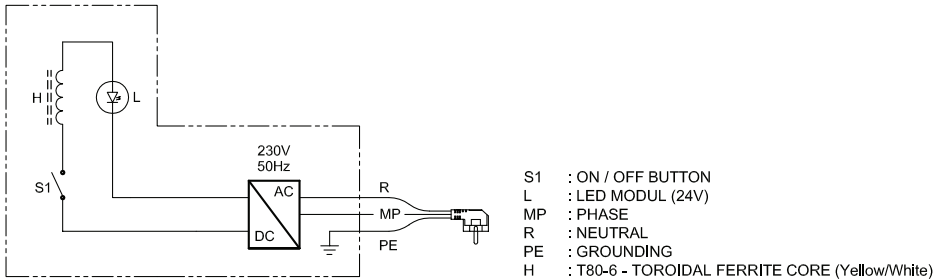
The Luxline Examination Light Device is made of metals and plastics and should be disposed of in accordance with the European directives 2011 / 65 / EU and 2012 / 19 / EU.

Upon end of life, the device should be disposed in accordance with local WEEE regulations and should not be disposed together with the general and contaminated medical device waste.

7. Technical Information

7.1. Electric Circuit Diagram

LUXLINE 45 / LUXLINE 45R / LUXLINE 45D

7.2. Technical Information

Mounting Type	Spider leg w/5 wheels
Stand Height	1300 mm
Lamp Head Arm	Spring arm, 400 mm length
Light Movement Diameter	1200 mm
Lamp Head	Led, Rotates 300°
Lamp Head Diameter	100 mm
Electricity Data:	
Rated input voltage	220-240 VAC
Frequency range	50/60 Hz
Power consumption	18 W
Photometric Values:	
Centralized illumination at a distance of 0,5 m Ec	65.000 lx ± %5
Bright field diameter at a distance of 0,5 m d10	Ø = 200 ± 30 mm
Bright field diameter at a distance of 0,5 m d50	Ø = 100 ± 20 mm
Colour temperature	3750 ± 250K
Color rendering index Ra	>97 CRI
Ambient conditions for transport, storage and use:	
Ambient temperature (storage and transport)	-40 °C / +85 °C
Ambient temperature (use)	-40 °C / +65 °C
Relative humidity (non-condensing) (storage and transport)	maks. %95
Relative humidity (non-condensing) (use)	maks. %75
Mode of operations:	
Permanent use	
Classification:	
Classification as per 93/42/EEC -Annex IX (medical device classes)	Class I
Electrical safety test and EMC	EN/IEC 60601-1, EN/IEC 60601-2-41, EN/IEC 60601-1-2
Lifetime of light source:	
Led operating time	50.000 h

The performance values are the values of the product powered by 230VAC 50Hz mains. If the product is powered with different values than those of the mains, deviations in performance values can be seen.




7.3. Residual Risks

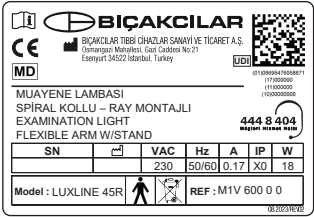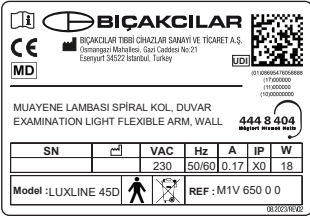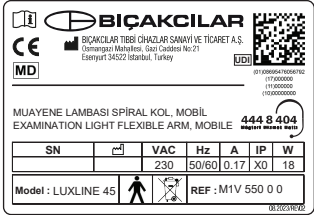The risk analysis carried out by the manufacturer's Risk Management Team has determined that the residual risk is as low as reasonably possible.

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